



CAUTION : Refer to the ultrasound system user manual for important probe care and cleaning instructions.

ACHTUNG : Wichtige Hinweise zur Pflege und Reinigung von Schallköpfen finden Sie in der Gebrauchsanweisung Ihres Ultraschallgeräts.

ATTENTION : Se reporter au manuel de l'utilisateur de l'échographe pour des instructions concernant la maintenance et le nettoyage de la sonde.

CUIDADO : Consultar el Manual del Operador del sistema de ultrasonido para obtener las instrucciones importantes relativas al mantenimiento y limpieza de la sonda.

CUIDADO : Consultar o Manual do Usuário do sistema de ultra-sonografia para obter as instruções importantes relativas à manutenção e limpeza do transdutor.

CAUTELA : Per le istruzioni relative alla manutenzione e pulizia delle sonde fare riferimento al manuale per l'utente del sistema ultrasuoni.

小心 : 请参考超声波系统用户手册，了解详细的有关探头维护和清洗方面的指导信息。

注意 : 重要なプローブの手入れ並びにクリーニング手順に関して超音波システムのユーザーマニュアルを参照して下さい。

		i12L	i12L-RS	3V	4C 4C-RS	4S	M4S M3S	348C gray 3.5CS 3.5C	3S 3S-RS	3S-LC	3SR	5S, 5S-RS 7S, 7S-RS 10S, 10S-RS	5S-LC	8S	10LB	8L 8L-RS		
	GE Ultrasound Contact Gel	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
	Aquasonics 100	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
	Clear Image	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
	Scan	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
	Sonogel	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	—
	T-Spray	⊗	⊗	⊗	⊗	OK	OK	OK	OK	OK	OK	⊗	⊗	⊗	⊗	OK	OK	< 20 minutes
	T-Spray II	OK	OK	⊗	⊗	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 20 minutes
	Asepti-Wipes	⊗	⊗	⊗	⊗	⊗	⊗	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 30 minutes
	Sani-Cloth Germicidal	⊗	⊗	⊗	⊗	⊗	⊗	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 30 minutes
	Sani-Cloth HB	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 30 minutes
	PeraSafe	⊗	⊗	⊗	⊗	⊗	OK	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 15 minutes
	Cidex	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
	Cidex OPA	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 15 minutes
	Cidex Plus	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 25 minutes
	Gigasept	⊗	⊗	⊗	⊗	⊗	⊗	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 30 minutes
	Gigasept AF	⊗	⊗	⊗	⊗	⊗	⊗	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 30 minutes
	Gigasept FF	⊗	⊗	⊗	⊗	⊗	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 30 minutes
	Metricide 14	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
	Metricide 28	OK	OK	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 50 minutes
	Metricide 30	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	⊗	< 25 minutes
	Ster-Bac Blu	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
	Wavicide-01	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	OK	< 50 minutes
		CE 0459	CE 0197	CE 0843	CE 0843	CE 0459	CE 0843	CE 0459	CE 0843		CE 0459	CE 0843		CE 0459	CE 0197	CE 0843		

