



Ultrasound Scanning Gel

- Product Description**
- Clear Image transcutaneous ultrasound scanning gel couples diagnostic and therapeutic ultrasound between the medical imaging equipment and the patient
 - Clear Image is formulated for optimal sound velocity and acoustic impedance to couple medical ultrasound
 - Clear Image is air-bubble free to reduce reflection and refraction as the sound waves pass through the gel
 - No dyes or perfumes, non-staining
 - Water-based, water-soluble formula
- Unit Dose Flexible Packaging**
- Prevents cross-patient contamination, eliminates nosocomial infection potential from refilling and reusing ultrasound gel bottles
 - Reduces institutional liabilities
 - Economical packaging
 - Ease of use
 - Reduces plastic waste up to 57%
- Use**
- Gel may be warmed before use; place Clear Image Singles upright in warmer
 - Apply transcutaneously and/or to the transducer prior to scanning
 - Remove with a dry and/or wet paper or cloth towel
- Sterility**
- Non-sterile; for transcutaneous use only
- Probe Compatibility**
- Clear Image does not contain oils, alcohol, fragrance, or other ingredients known to harm probe face materials
 - Approved by major diagnostic ultrasound equipment manufacturers
- Contraindications**
- Not intended for use as an electrode conductor
 - Not intended for use in invasive ultrasound procedures
- Storage**
- Store away from direct sunlight to preserve viscosity
 - Recommended storage temperature: 32°F to 90°F (0°C to 32°C)

Ordering Information

Clear Image Singles

<i>Container</i>	<i>Packet Size</i>	<i>Catalog #</i>
4 cartons / 100 pkts. each	1 oz. (30 mL)	02-400B-030
Refill case / 400 packets	1 oz. (30 mL)	02-400-030
4 cartons / 100 pkts. each	0.5 oz. (15 mL)	02-400B-015
Refill case / 400 packets	0.5 oz. (15 mL)	02-400-015

Clear Image Singles Lite

<i>Container</i>	<i>Packet Size</i>	<i>Catalog #</i>
4 cartons / 100 pkts. each	1 oz. (30 mL)	06-400B-030
Refill case / 400 packets	1 oz. (30 mL)	06-400-030

